



Safety Visor



180°
Optical Screen



Airflow circulation
No fog



Glasses
compliant



Light weight
85 grams



Recyclable



EN 166
UL94-V0
REACH



- Full facial protection
- Perfect fit adjustable Velcro strap-closure
- Full 180-degree visibility & high optical transparency
- No fog & high breathability through airflow circulation
- Ultimate comfort & extremely light weight
- Compatible for people wearing glasses
- Handmade in Europe by conTeyor

Technical specifications conTeyor Face Mask

- Full facial protection from the middle of the forehead to underneath the chin area
- One-size-fits-all
- Optical screen by means of a high-quality transparent PVC
- PVC is sustainable and recyclable
- High operational and ergonomic comfort
- Extremely light weight: 85 grams
- Detachable shield, easy to clean and disinfect
- Size shield: 36,5 x 23 cm – Size strap: 60 x 2 cm
- Complies with EN 166, UL94-V0 and REACH

Why conTeyor?

Our core business is sustainable packaging, so no waste!

We decided to convert a part of our textile production for the manufacturing of masks and visors. We are proud to be part of the global battle against COVID-19 and to support our communities.

Important user information

1. Recommended for non-medical, personal use
2. For hygienic purposes: disinfect the visor after every usage

How to use the visor correctly

1. Wash and disinfect your hands thoroughly before using the visor
2. Take the strap and the shield and connect them together using the Velcro as shown on the picture
3. Your visor is mounted
4. Open the Velcro strip and position it around your head
5. Adjust the visor and close the Velcro strip
6. To remove the visor, open the Velcro strip without touching the front of the visor
7. After removing the visor, wash and disinfect your hands
8. Avoid touching the visor while using it, if you do, wash or disinfect your hands thoroughly

Maintenance instructions

Clean the face shield by applying disinfectant cleaner on a light cloth and by making light circular movements

Do not expose to high temperature

Do not apply high pressure



Product information conTeyor Safety Visor EU – Eng – v2

Disease prevention



DISCLAIMER: The use of this product is for non-medical purposes only and strictly at the individual's own risk. The level of effectiveness depends on its use and circumstances. Considering the urgent need on short notice globally, the development process did not allow for extensive field testing and certification. The manufacturer assumes no liability for the use or misuse of this product.