





# Safety Visor

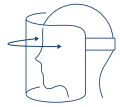

180° Optical Screen

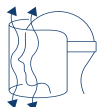

Airflow circulation  
No fog



Glasses  
compliant



Light weight



Recyclable



EN 166  
UL94-V0  
REACH



- Full facial protection
- Perfect fit adjustable Velcro strap-closure
- Full 180-degree visibility & high optical transparency
- No fog & high breathability through airflow circulation
- Ultimate comfort & extremely light weight
- Compatible for people wearing glasses
- Handmade in North America by conTeyor

## Technical specifications conTeyor Face Mask

- Full facial protection from the middle of the forehead to underneath the chin area
- One-size-fits-all
- Optical screen by means of a high-quality transparent PETG
- PETG is environmentally friendly and non-toxic
- High operational and ergonomic comfort
- Extremely light weight: 3oz
- Detachable shield, easy to clean and disinfect
- Size PETG part: 14.4' x 9' – Size strap: 23.8' x 0.8' & 19.5' x 0.8'
- Complies with EN 166, UL94-V0 and REACH

## Why conTeyor?

Our core business is sustainable packaging, so no waste!

We decided to convert a part of our textile production for the manufacturing of masks and visors. We are proud to be part of the global battle against COVID-19 and to support our communities.



### Important user information

1. Recommended for non-medical, personal use
2. For hygienic purposes: disinfect the visor after every usage

### How to use the visor correctly

1. Wash and disinfect your hands thoroughly before using the visor
2. Take the strap and the PETG part and fix it to the Velcro as shown on the picture
3. Your visor is mounted
4. Open the Velcro loop and position it around your head
5. Adjust the visor and close it
6. To remove the visor, grab it by the Velcro strip without touching the front of the visor
7. After removing the visor, wash and disinfect your hands
8. Avoid touching the visor while using it, if you do, wash or disinfect your hands thoroughly

### Maintenance instructions

Clean the face shield by applying ethanol on a light cloth and by making light circular movements.

Do not expose to high temperature.  
Do not apply high pressure.



Product information conTeyor Safety Visor NA - Eng - v1

### Disease prevention



DISCLAIMER: This product was designed as a face cover and is not designed to be used by healthcare professionals. This cover is not a 'personal protective equipment' (PPE) as defined in Regulation (EU) 2016/425 of the European Parliament and of the Council or 'medical device' as defined in Regulation (EU) 2017/745 of the European Parliament and of the Council. The use of this product is for non-medical purposes only and strictly at the individual's own risk. The level of effectiveness depends on its use and circumstances. Considering the urgent need on short notice globally, the development process did not allow for extensive field testing and certification. The manufacturer assumes no liability for the use or misuse of this product.

conTeyor Packaging Solutions, S.A. de C.V., Av. Valle de San Isidro #530, Bosques de San Isidro, 45133 Zapopan, Jalisco, México  
conTeyor US Inc., 1801 Michael St., Madison Heights, MI 48071  
[www.conteyor.com](http://www.conteyor.com)